

JATI AUTOMASI ENTERPRISE

(Co. no 001773550-P)

No. 9B Jalan SG ¾, Taman Sri Gombak, 68100 Batu Caves, Selangor Darul Ehsan.

Tel : 03-61859326 Fax : 03-61871294

BABY RESUSCITATION C/W RADIANT HEATER MODEL DC 4100

- The Radiant Heater provides a safe, effective body temperature control system for newborn infants. A stable and even heat irradiation by means of the long wave infrared radiant ceramic heater, directed towards the treatment table area to warm the infant.
- Immediate warming feasibility on simultaneously switching on the temperature heating control to a desired temperature up to 38 degree C. in any environmental room temperature.
- Trolley consists of tilting top with Acrylic panels' with fold down type (non detachable)
- 50mm PVC covers with PU Foam.
- A full-length drawer, open shelf, 2 half drawers. A bracket for oxygen cylinder & a shelf at top for accessories - a stop clock provided.
- 3 position tilt with 12° by ratchet at every 4° position
- Mobile on 4 x 125mm swivel castors with brakes diagonally fitted
- **Overall Size:** 760mmL x 460mmW x 1670mm – 1870mmH (w/o Bumper)

ROOM TEMPERATURE 26 degree C (Air-Con Room)

To obtain 36 degree C temperature: -

- 1) Height of hood from mattress / Baby 60cm – 80cm
- 2) Temperature setting of Dial knob 3

Thermostat adjustment for temperature

Level	Temperature
On	26°C to 28°C
1	27°C to 29°C
2	28°C to 33°C
3	30°C to 40°C
4	35°C to 46°C
Full	50°C to 80°C

ROOM TEMPERATURE 31 degree C temperature: -

To obtain 36 degree C temperature:-

- 1) Height of hood from mattress / Baby 60cm-80cm
- 2) Temperature setting of Dial knob 1.5

